

Mepitac®



SECURE AND GENTLE FIXATION TAPE WITH SAFETAC® TECHNOLOGY

Mepitac® tape is atraumatic to the skin upon removal and is ideal fixation for patients with fragile skin or at risk for those reactions to traditional tapes and adhesives.

Safetac® technology layer


- Reduces pain and trauma upon removal
- Leaves no residue upon removal

Polyurethane backing film

- Breathable
- Moisture proof

Knitted fabric

- Conformable
- Easy to cut

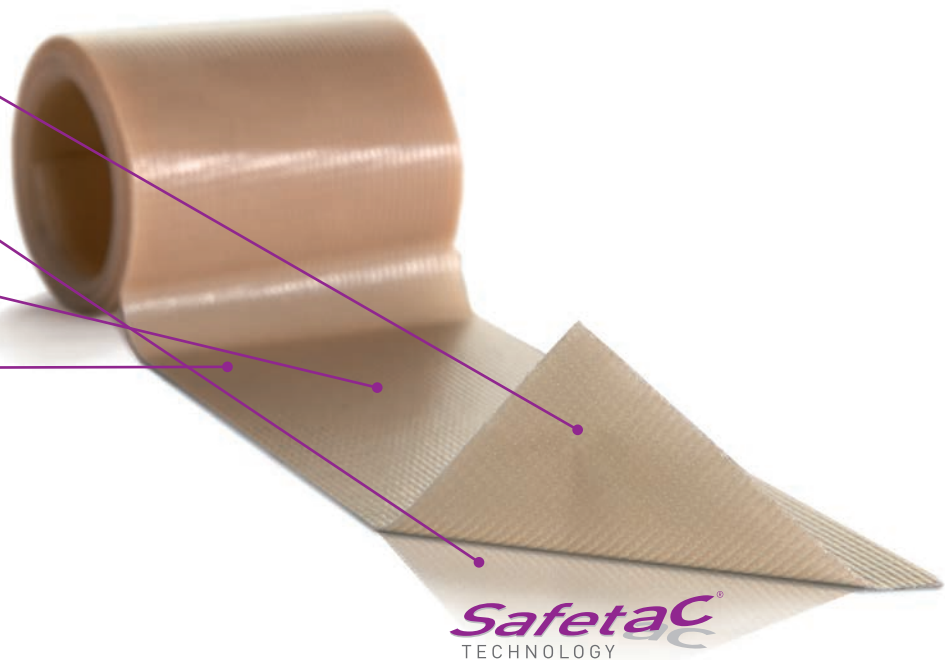


Safetac® technology


Safetac® technology. Less pain and less trauma.

Safetac® is a patented adhesive technology that minimizes pain to patients and trauma to wounds. Safetac® technology is available exclusively on Mölnlycke Health Care dressings, including Mepilex® wound care dressings, Mepitel®, Mepiform® and Mepitac®.

For more information visit www.molnlycke.us

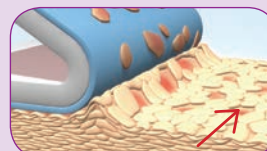


Painful skin stripping with traditional adhesive

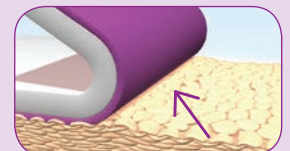


No skin stripping with Safetac® technology


How to use Mepitac®

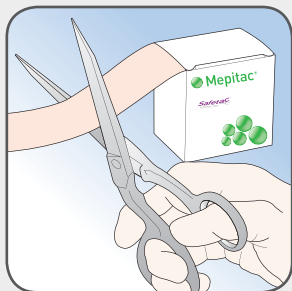

Make sure the skin is clean, dry and free from any ointment, etc. Cut off the desired length of Mepitac® using scissors.

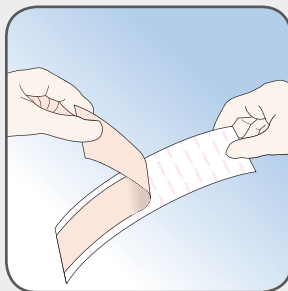

Remove the protective release film.

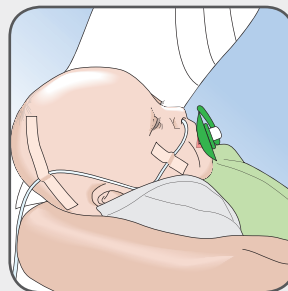

Apply Mepitac® without stretching. If inspection or repositioning is necessary, Mepitac® can be lifted and reapplied.

How Mepitac® works

Mepitac® is highly conformable and easy to apply and remove. The Safetac® layer ensures secure fixation and atraumatic removal from skin and hair. If inspection or repositioning is necessary, Mepitac® can be lifted and re-applied. It is an ideal fixation solution for patients with fragile and sensitive skin or with allergies to traditional adhesives. Mepitac® is suitable for patients requiring repeated application and removal of tape over the same area (e.g., neonates and dialysis patients).

Benefits of Mepitac®

- Atraumatic to the skin on removal
- Conforms well to body contours
- Secure fixation
- Leaves no residue
- Breathable, moisture proof film backing
- May be repositioned
- Suitable for patients with fragile skin, sensitive skin or allergies to traditional adhesives




Areas of use

- Fixation of medical devices such as drains, tubes, probes, electrodes, IV cannulae, and dressings
- Skin protection when used under devices (e.g., tubes).
- May be repositioned

Note:

The adhesion of Mepitac® may decrease when used in humid conditions such as incubators or on patients who are diaphoretic. Mepitac® is not suitable for fixation of arterial catheters and arterial cannulae.

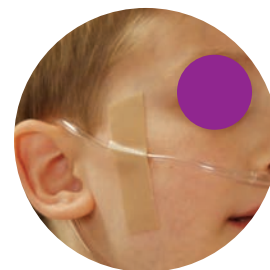

Mepitac® Ordering Information

Packaged sterile in single rolls.

Product Code	Size	Rolls/box	HCPCS
298300	3/4" x 118" (2 x 300 cm)	12	A4452
298400	1.5" x 59" (4 x 150 cm)	12	A4452

Safetac®
TECHNOLOGY

5550 Peachtree Parkway, Suite 500, Norcross, GA 30092 | 800.882.4582 | www.molnlycke.us | www.mhcwoundcare.com

The Mölnlycke Health Care, Mepilex, Mepitel, Mepiform, Mepitac, and Safetac, names and respective logos are registered trademarks of Mölnlycke Health Care AB. Distributed by Mölnlycke Health Care US, LLC, Norcross, Georgia 30092. © 2012. Mölnlycke Health Care AB. 1.800.882.4582. US038611010

MÖLNLYCKE®
HEALTH CARE

MHC-2012-XXXX - 270233 07.12